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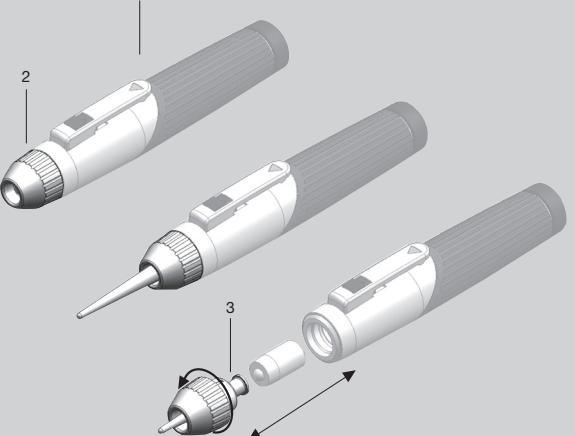
Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further exclude any warranty claims which result from such a change or modification. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For repairs, please contact your supplier, who will return the goods to us or our authorized agent.

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Instructions for use

FRANÇAIS

Read these instructions carefully before using the HEINE mini 3000® Cliplamp and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

Safety in use

- Do not use alkali manganese batteries containing lithium as bulb life will be reduced.
- The performance of this instrument can only be guaranteed if genuine HEINE Accessories are used.
- Do not shine the light directly into the patient's eyes.
- The HEINE Ear Light is used by hearing aid specialists or trained personnel as a device for the anatomical adjustment of the geometry of hearing aids.
- The HEINE Ear Light is not suitable for the examination of the eardrum and may not be introduced too deeply into the acoustic meatus.
- Inspection / Functional testing:
The HEINE Ear Light must be checked for its functionality for its intended use on a regular basis. The item must be discarded, if any damage is visible or suspected.

HEINE mini 3000® Battery Handle (1)

Range of applications:

The HEINE mini 3000® battery handle should only be used for operation to power HEINE mini 3000® instruments.

Batteries:

- 2 alkali cells (AA/ LR6)
Important: Do not use alkali manganese batteries with lithium as bulb life will be reduced.
- The HEINE mini 3000® battery handle can be charged in the HEINE mini NT® with the HEINE mini 2Z rechargeable battery 2,5 V.

Charging time:

- The mini NT indicates when the rechargeable battery is fully-charged. The illuminated ring is lit continuously.
• mini 2Z rechargeable battery (2,5 V, NiMH) = ca. 4 h (730 mAh)

Instrument connector:

Internal thread in handle head.

Switching on:

Push switch down.

Switching off:

Push switch up. If the handle is put back firmly into a coat pocket the handle is automatically switched off.

Changing the batteries:

Unscrew the end-cap and shake out the old batteries/rechargeable battery. Insert new batteries/rechargeable battery with the plus contact facing the handle head.

HEINE mini 3000® Cliplamp (2) and Ear Light (3)

Range of applications

- HEINE mini 3000® Cliplamp:

The HEINE mini 3000® Cliplamp is designed solely for the examination of the surface of the skin.

- HEINE Ear Light:

The HEINE Ear Light in combination with the HEINE mini 3000® Cliplamp is exclusively designed for the lighting of the transition area from the auricle (external ear) to the external acoustic meatus (ear canal).

Cleaning and Maintenance

The HEINE mini 3000® Cliplamp is not suitable for treatment in the ultrasonic bath, for sterilisation, spray disinfection or treatment by immersion in liquids. The surface of the HEINE Ear Light may be damaged by alcoholic cleaning agents.

- Manual Cleaning:

Use a damp cloth for cleaning the HEINE mini 3000® Cliplamp and HEINE Ear Light.

- Disinfectant:

The HEINE mini 3000® Clip Lamp can be disinfected by wiping with a disinfectant which is approved for use on medical devices made of plastic. Follow the instructions of the disinfectant manufacturer.

Important note: Disinfection by disinfectant spray or soaking and sterilisation are not approved and will lead to damage to the instrument.

Important note: Any attempt to disinfect the Ear Light will lead to material damage and increase the likelihood of breakage.

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