boso

classic, classico

- Ⓓ Gebrauchsanweisung
- Ⓕ Mode d'emploi
- Ⓘ Istruzioni per l'uso
- Ⓖ Instructions for Use
- Ⓔ Instrucciones de Uso

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Instructions for Use

Introduction:

This blood pressure monitor has been manufactured to the specifications of the European Directives as set out in the law on Medical Products (CE Mark), as well as to the European Standard EN 1060, Part 1 »Non-Invasive blood pressure instruments – general requirements« and Part 2 »Additional requirements for mechanical blood pressure instruments«.

Regular calibration testing is to be carried out in accordance with the regulations existing in individual countries.

Important:

Protect your blood pressure monitor from damp and dirt. Care has to be taken not to knock or drop the unit and that the cuff does not get damaged by sharp objects (needles, scissors etc.).

Do not inflate to above 300 mmHg.

Individual measurements should not exceed a maximum of 2 minutes.

Between measurements there should be an interval of at least 2 minutes.

Placing the cuff

Place the cuff so that the lower cuff edge finishes about 2 – 3 cm above the elbow and the centre of the rubber bag (see marking) on the artery.

Taking the measurement:

Fully close the valve by turning the air release screw to the right. Inflate the cuff to above the systolic pressure. By turning the air release valve the air release speed can be adjusted. The World Health Organisation (WHO) recommends a deflation rate of 2 – 3 mmHg per second.

After the measurement is completed open the release valve to fully deflate the cuff.

Cuffs:

Only use the prescribed boso cuffs with their special connector.

Technical Data:

Measuring range: 0 – 300 mmHg

Accuracy of indication of pressure: ± 3 mmHg

Storing conditions:

–30°C – 70°C, 15 – 85% rel. humidity

Operating conditions:

10°C – 40°C, 15 – 85% rel. Humidity

Cleaning of unit and the cuff:

- To clean your unit it is only necessary to use a soft dry cloth.
- To clean the cuff, first remove the rubber bag. The outer cuff can then be washed by hand at a maximum temperature of 30 degrees Celsius.

Disinfection:

For disinfectant wipes (at least 5 minutes exposure time) of the device and the cuff, we recommend the disinfectant liquid Microzid Liquid Sensitiv (Schülke & Mayr).

To disinfect the velcro of the cuff, we recommend spray disinfection.

Guarantee / Service:

For details on the conditions of guarantee, please refer to the guarantee card supplied.

Any guarantee is valid only if the item for return is accompanied by the relevant guarantee card duly completed, signed and stamped by the authorised dealer.

For both guarantee and repair work, please send the instrument carefully packed and postage paid either to your authorized dealer or directly to:

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Bosch + Sohn GmbH u. Co. KG
BOSCH + SOHN GmbH u. Co. KG